

# Silicone dressings CEREDERM®

*"An effective  
solution for  
reducing scars"*

TREAT AND  
PREVENT  
HYPERTROPHIC  
AND KELOID SCARS


# WHAT IS CEREDERM®?

Transparent and easy to use, CEREDERM® is a self-adhesive or gel silicone dressing for treatment and prevention of hypertrophic (red coloured area) and keloids (fibrous, firm and thick bulge) scars. The silicone dressing softens the scar and reduces its redness. The mechanical effect of the auto-adhesive dressing smoothes out the scar and limits its development. Its semi-occlusion effect gets the scar appearance better.

All components in our CEREDERM® dressings have been carefully selected for their biocompatibility.

## THE RANGE OF CEREDERM® DRESSINGS IS AVAILABLE

### IN 6 SELF-ADHESIVE SHAPES (BOX OF 10)

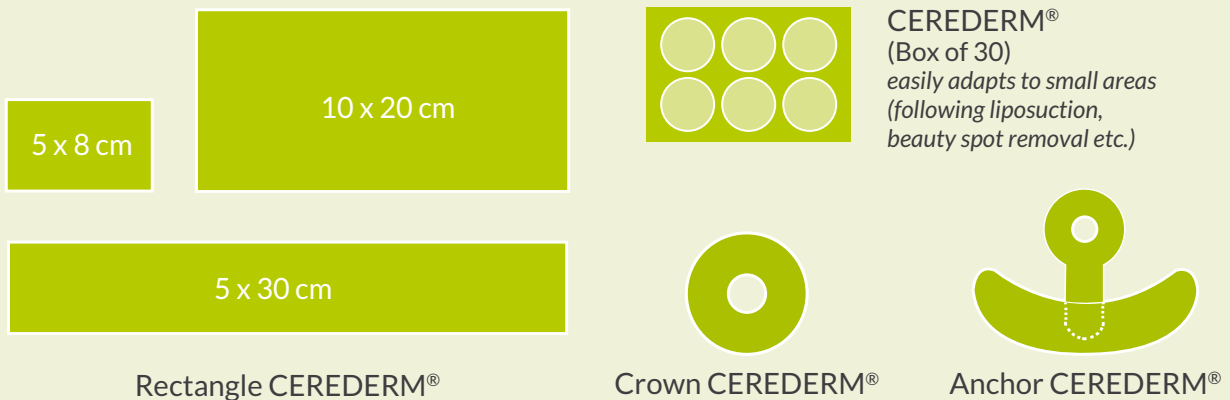

### AS A GEL



15g tube  
and **50g tube**



### INDICATIONS

Treatment and prevention of hypertrophic and keloid scars of any origin.



### CONTRAINDICATIONS

- › Membranes
- › Open wound
- › Unhealed wound
- › Scar near the eyes

*In case of skin reaction, stop the application and consult your doctor..*



### RESULTS

Visible after 2-3 months of use

Softer, smoother and less red, the scar is visibly improved.

# CEREDERM® SELF-ADHESIVE SILICONE DRESSING

## RECOMMENDATIONS

Wear each dressing until it loses its adhesion, and during maximum 3 days (preferably 24 hours a day).

For strictly personal use only (single patient use).

Recommended treatment duration: 2 to 6 months.

## INSTRUCTIONS FOR USE

Remove the CEREDERM® dressing from its packaging. Cut it to the required size, making sure it is at least 1 cm larger than the wound on all sides.



Carefully clean and dry the area before applying the dressing. Place the adhesive side of the dressing on the closed wound, and gently remove the protective film, without pulling.



CEREDERM® dressings can be used on their own without the need for cream, although you may also use a supportive device if required (elastic band or compression garment). The dressing is water-resistant and repositionable.



**WARNING,  
 CEREDERM®  
 SELF-ADHESIVE  
 DRESSINGS  
 DO NOT PROTECT  
 FROM UV RAYS.**

## CEREDERM® is:

- ✓ **Easy to cut:** fits to the size of the wound.
- ✓ **Soft:** adapts perfectly to the body shape.
- ✓ **Painless:** can be worn all day long.
- ✓ **Thin & transparent:** can be worn discretely.
- ✓ **Self-adhesive:** holds itself in place.
- ✓ **Washable:** the same dressing can be worn for 2-3 days.
- ✓ **Effective:** can be used alone, no need for cream.

- ✓ EFFECTIVE
- ✓ PAINLESS
- ✓ INVISIBLE

# CEREDERM® GEL SILICONE GEL DRESSING

- › **Practical:** dries rapidly and can be covered with make-up.
- › **Flexible:** perfect for uneven surfaces, the face, joints and body folds.
- › **Painless:** easy to apply and pain free.
- › **Aesthetic:** transparent and odourless.
- › **Effective:** can be used alone, no need for cream.

## RECOMMENDATIONS

Apply CEREDERM® GEL twice daily (morning and evening).

For external use only.

Recommended treatment duration: 2 to 6 months.



**WARNING, CEREDERM® GEL DOES NOT PROVIDE UV PROTECTION.**

## INSTRUCTIONS FOR USE



Inspect the scar and **clean it at least once a day with clean water** and pH neutral soap.



**Dry the skin well** before applying the gel.



**Apply a thin layer of gel,** in a firm, circular motion, moving along the scar. Allow a few minutes to dry.



**Close the tube firmly** after use.

